

THORAGUARD®

INTELLIGENT SURGICAL DRAINAGE

CHEST TUBE KIT SETUP INSTRUCTIONS

SOAK CHEST TUBE PRIOR TO USE & KEEP VALVE DRY



Place Chest Tube
in Patient

Cut Chest Tube
After Placement

Connect
Chest Tube to
SmartValve



Follow Drainage Kit
Setup Instructions to
Attach Chest Tube
to Drainage Tube

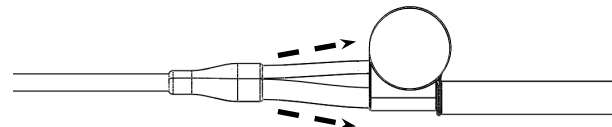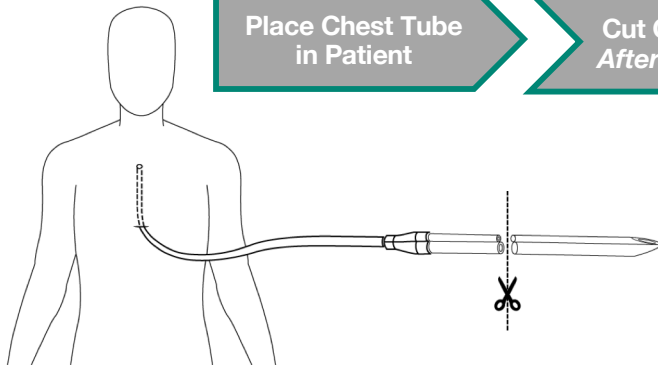

NOTE: Refer to OPERATOR'S MANUAL for instruction on system-level use

IFU-1570 Ver 5 07-01-2020